



Better Sound Limited


Phone: 020- 7482 0177 Fax: 020- 7482 2677

31 Cathcart Street, London, NW5 3BJ

e-mail: admin@betersound.co.uk

Phonak Repeater Guide/FAQ


1. On/Off button – push & hold > 3 seconds
2. Status indicator light(s)
3. Connect button (for Roger earpieces only)
4. Micro-USB power/charging socket

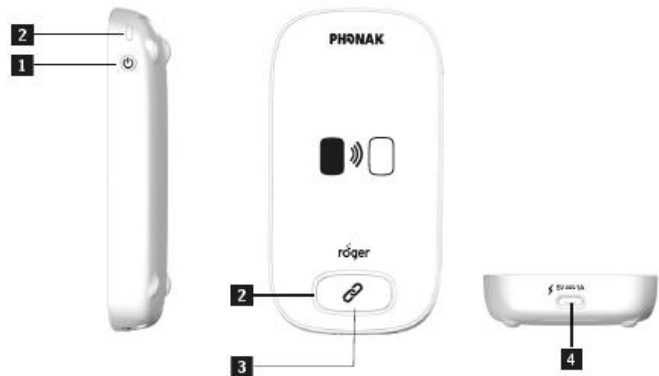

Pairing Roger Repeater with transmitter


N.B. Better Sound sends out kits with the pairing already completed, so the kit is ready for use straight out of the box.

To connect a Roger Repeater to a Roger BaseStation or a Roger Touchscreen Mic for the first time, hold it within 10 cm of the transmitter (TX) and briefly press the Connect button on the front panel (of the TX). Keep any other Roger devices out of range during this process. An unlimited number of Roger Repeaters can be connected. Pairing of a Roger Repeater is only necessary the first time it is used. The Roger Repeater will then remain connected with the Roger TX, even if either device is restarted.

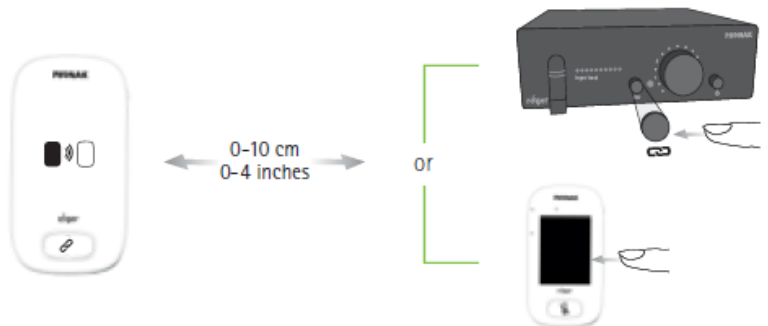

Pairing Roger Earpiece with Roger BaseStation/Touchscreen Mic/Repeater

To connect a Roger Earpiece for the first time, hold it within 10 cm of the TX/Repeater and briefly press the Connect button on the front panel. Only one Roger Earpiece can be paired at a time. Keep any other Roger devices out of connection range during this process. An unlimited number of Roger Earpieces can be connected. Pairing of a Roger Earpiece is only necessary the first time it is used. The Roger Earpiece will then remain connected with the Roger TX/Repeater, even if either device is restarted.

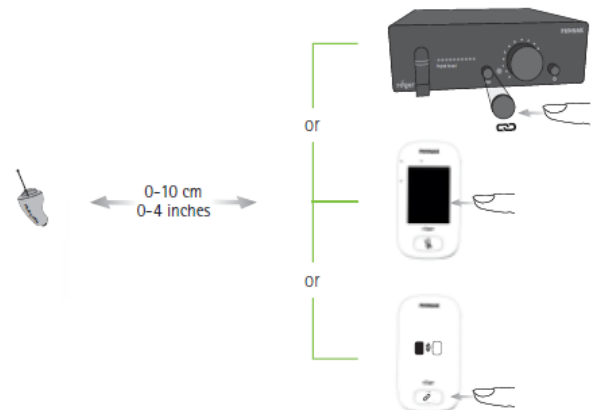

For optimum Repeater placement see guide to status lights – Audio broadcast range (overleaf)

The Roger Repeater extends the range of Phonak's Roger Earpiece
An unlimited number of Repeaters may be used

Guide to Repeater indicator status lights



On/Off
 Connecting
 Repeater or receiver connected

No device to connect in range

Not a compatible device

Audio broadcast range

Not at optimum range from TX. Repeater can be sited further away if desired.
 OK - optimum siting of Repeater achieved
 too far
 out of range



On/Off
 Charging
 Fully charged
 Switched on
 Recharge device

To RESET the Roger Repeater

Push & hold the On/Off and Connect button > 8 seconds
It will then be necessary to pair the Repeater with a TX again



Roger Repeater Specification:

Battery life: 6-8 hours

Charging time: 2 Hours

Operating distance/range: Between Roger BaseStation/Touchscreen Mic/Repeater units - **up to 50m**
Roger BaseStation/Touchscreen Mic/Repeater(s) and a Roger earpiece – **10-30m**

Repeater can be permanently powered via USB